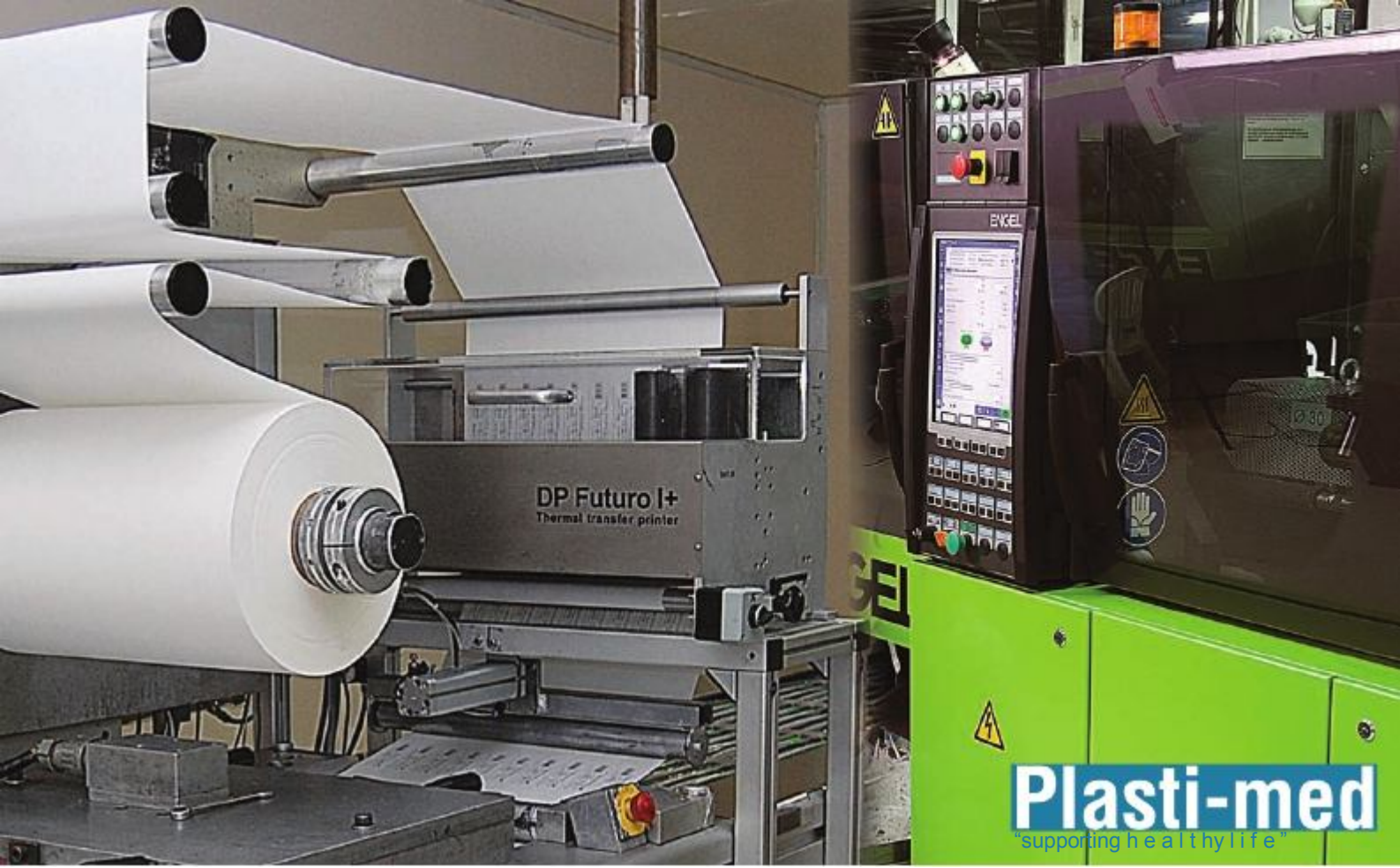
SUCTION & DRAINAGE  
SYSTEMS

**Plasti-med**  
"supporting healthy life"

ASPIRASYON & DRENAJ  
SİSTEMLERİ  
2012

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### HISTORY

The company was formed in 1996, initially specialising in Anesthesiology. Plasti-med has since grown dramatically – expanding the range of products into the specialist areas of Gynecology, Drainage and General use disposable products. More recently, Plasti-med has expanded its expertise and product range into the field of Urology. A dynamic and inventive company, Plasti-med has enjoyed international success, and its global operations stretch around the world. Plasti-med is proud to announce that it has recently celebrated its 15th anniversary and hopes to continue to go from strength to strength in the next 15 to come.

### QUALITY POLICY

Our goal is to continually supply high quality medical devices to healthcare professionals the world over and to ensure that these products fully comply to both international and national regulatory and customer requirements. All our devices are manufactured in compliance to the Council Directive 93/42/EEC, 2007/47/EC.

### PRODUCTS

A specialist in Anaesthesiology products, Plasti-med has gained an ever increasing share in the marketplace for over 15 years. With extensive experience in the design, development and manufacture of premium quality medical devices, the product range continues to grow. All products supplied by Plasti-med undergo strict quality control to ensure complete customer satisfaction. Plasti-med commits to Research and Development of new products, in addition to the current wide range on offer – aiming to be at the forefront of technological advances. Our Customer Service department is on hand to offer further details on any of our products – for more information please contact us.

### STANDARTS

Plasti-med has long been accustomed to operating in a highly regulated environment with regards to design and development of medical products. All Plasti-med devices are manufactured to the highest possible standards and have been bearing the CE mark since 2003 – demonstrating compliance with the Medical Device Directive 93/42/EEC, 2007/47/EC. The Plasti-med Quality Management System is certified according to and regularly audited against ISO 9001:2008 & ISO 13485 (2003). It is this commitment to Quality Assurance which enables us to serve and satisfy all our customers' expectations.

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\*Special products can be manufactured upon request

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## SYSTEM ADVANTAGES

- Suction Liners provides the protection from possible infections, which causes from hospital ambience.
- During working of user personel and crew, provides easy and practical application.
- It has easy connection and assembly.
- When it fills, it's easy to change.
- It removes spillage and springing risks.
- It helps to protection of hospital hygiene.
- It provides advantage to destroy controlled medical waste.
- Due to its closed system, it prevents the spread of odor to the other departments.
- Both operation and anaesthesia crews can use it during the surgical operation.
- Generally it can be use similar liquid suction processes which has vacuum sources.

## SUCTION LINER SPECIFICATIONS

- No contact with patient.
- Raw material and cover system are made of durable plastic.
- It's capacity is optional and 1200, 1500, 2000, 3000 ml.
- Liners are flexible and sole part is flat shaped.
- Suction end of patient port has anti-reflux valf, which prevents the back flow.
- It has a special filter in vacum suction end, which prevents the transition of bacterial and virus and stop the suction when liner filled.
- It has a cover system, which closes all the ports at the end of the process.
- It includes 1,8 m patient suction tube for suction process from the patient.
- It contains 2 pieces rubber plastic 1 piece kapkon connector for provide the connection between patient and vacum source.
- Aspiration Bag has wide sample port, which provides the protection of solid samples received similar bone and tissue samples from arthroscopy, endoscopy etc applications.

## ADDITIONAL MATERIALS

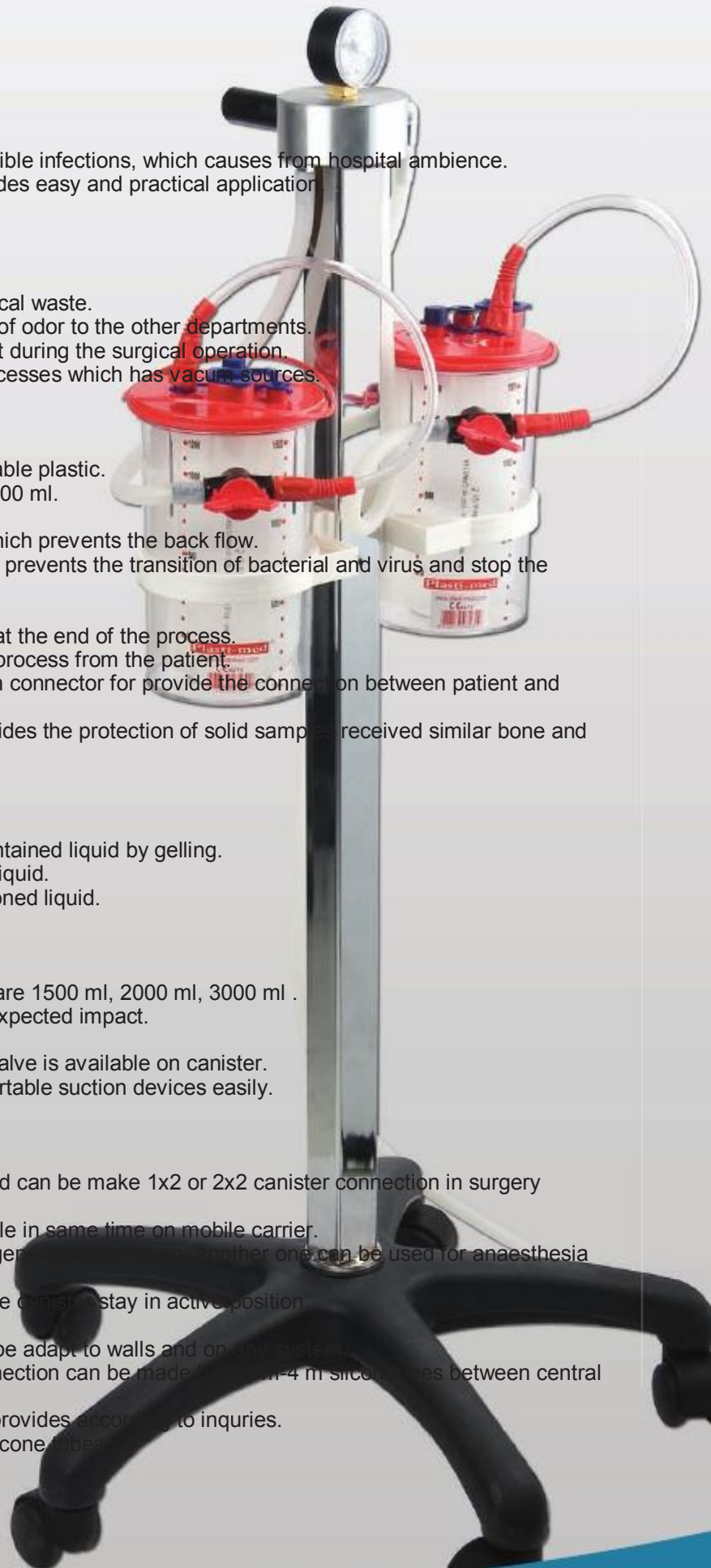
- **Gelling** : It provides the solidification of liner contained liquid by gelling.
- **Antifoam** : It prevents the foaming of suctioned liquid.
- **Disinfectant**: It provides the disinfection of suctioned liquid.  
*Additives are optional.*

## CANISTER SPECIFICATIONS

- It has three kinds of canisters, which capacities are 1500 ml, 2000 ml, 3000 ml .
- Canister is polycarbonat material durable for unexpected impact.
- It can be autoclavable up to 121 celcius degree.
- For easy bag handling, vacuum switch (on-off) valve is available on canister.
- It can be adapt for central vacum system and portable suction devices easily.
- It can be adapt to wall and rail.

## CONNECTION SPECIFICATIONS

- It can be connect with mobile carriage system and can be make 1x2 or 2x2 canister connection in surgery operation rooms.
- Connection done, at least 2 canister can be usable in same time on mobile carrier.
- In double system, 1 canister can be used for surgery team, another one can be used for anaesthesia team.
- During the parallel connection of 4 canister, all the canister stay in active position.
- Canisters can be connected as serial.
- During Intensive Care Units and services, it can be adapt to walls and on an over bed.
- During the use of Mobile Carriage Systems, connection can be made by using 4 m silicone hoses between central vacum system and canister. In special cases, solution provides according to inquiries.
- In case of montage to the wall, it uses 1-1,8 m silicone hoses.

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## Removal of the Bag from the Canister

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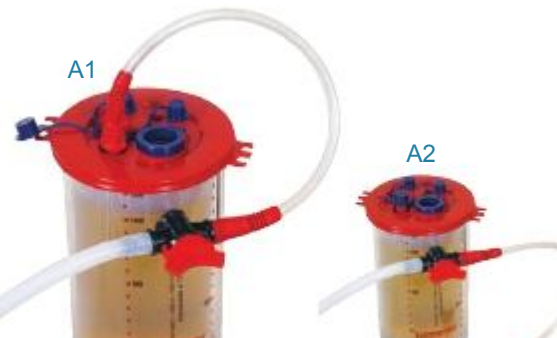
When is the suction bag is filled, the vacuum regulator and/or canister valve is turned off the 1,8 m patient tube is removed from the patient port of the bag.

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A1  
A2

The vacuum tube is removed from the vacuum port of the bag.

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You should close the port lid after removing the vacuum tube.

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Patient hose removes from patient port, which is on the suction collection bag. Close the port cover.

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You should use the handles to remove the bag from canister.

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The distributor manometer is fixed with a screw to the top part of the metal hanger.

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**TROLLEY MONTAGE MANUEL 1**  
**MOBİL ARABA KURULUM KILAVUZU**

During installation of (4 canister) mobile trolley, you should appropriately mountage the products no.160 213 05 and no. 160 213 01 by clamps to the iron holder by screw.

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**TROLLEY MONTAGE MANUEL 2**  
**MOBİL ARABA KURULUM KILAVUZU**

The canisters are placed in the clamp, both of the silicone tube ends are attached to the distributor and the canister.

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**TROLLEY MONTAGE MANUEL 3**  
**MOBİL ARABA KURULUM KILAVUZU**

Montage of two canisters trolley, mount 2 pieces of reference no: 160 213 02 plastic clamp to wheel carrier facing one another.

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Montage of four canisters trolley, mount 2 pieces of reference no: 160 213 02 and 2 pieces of ref no: 160 213 05 plastic clamp to wheel carrier facing one another.

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Montage of two canisters trolley, use vacuum distributor reference no: A AST 034

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Montage of four canisters trolley, use vacuum distributor reference no: A AST 035

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## Trolley Montage Manuel Mobil Araba Kurulum Kılavuzu

Vacuum tube is attached to the on-off switch of the canister.

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TROLLEY MONTAGE MANUEL 5  
MOBİL ARABA KURULUM KILAVUZU

The suction bags are placed in the proper canisters.

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TROLLEY MONTAGE MANUEL 6  
MOBİL ARABA KURULUM KILAVUZU

The red end of the vacuum tube is attached to a valve of the on-off switch of canister, the other end is attached to the vacuum port of the suction bag.

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TROLLEY MONTAGE MANUEL 7  
MOBİL ARABA KURULUM KILAVUZU

The patient tube is attached to the patient port by elbow part.

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Trolley Montage Manuel  
Mobil Araba Kurulum Kılavuzu

The central silicone vacuum tube is attached to the connection point of the distributor.

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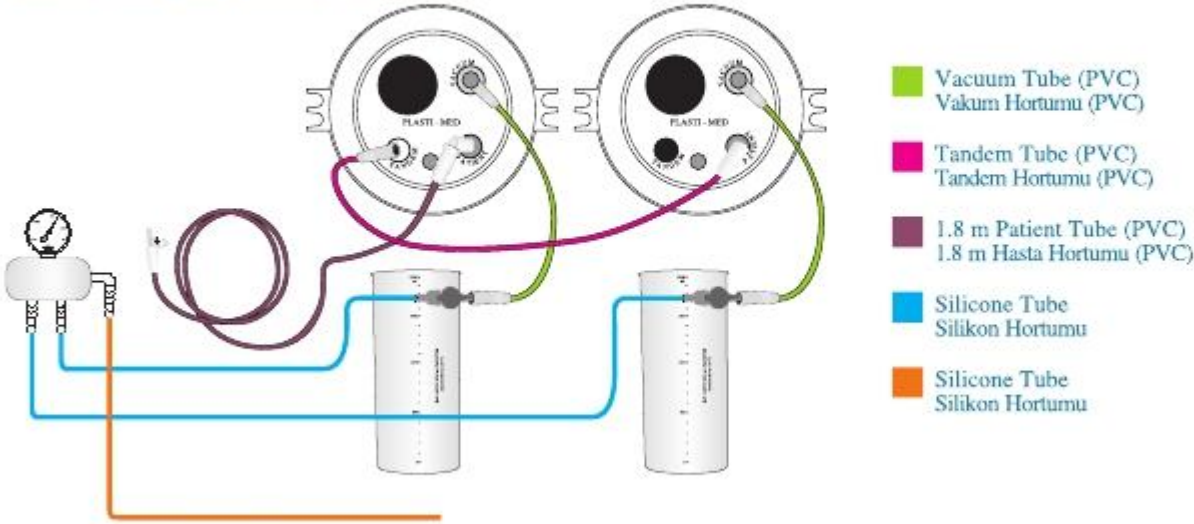
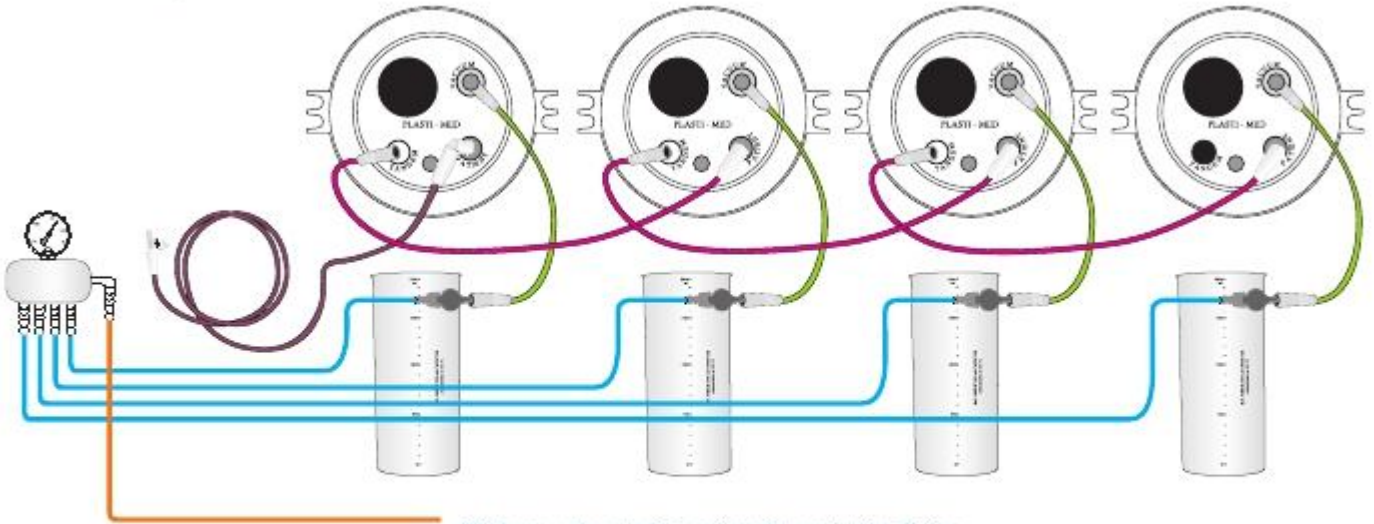
\*Colors on the graphic are just for easier installation.  
Original products are made of transparent white and silicone material.

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## Wound Drainage Set

Mini 30cc, 200 cc, 400 cc reservoir / Round drain from 6 ch to 18 ch  
 Single and double trocar / Cut-to-fit connector  
 For low pressure wound drainage

Mini 30 cc	Tek Trokar Single Trocar
Tip/Type	Ref No
6 ch	103 406
7 ch	103 407
8 ch	103 408

200 cc	Tek Trokar Single Trocar	Çift Trokar Double Trocar
Tip/Type	Ref No	Ref No
8 ch	103 208	103 108
10 ch	103 210	103 110
12 ch	103 212	103 112
14 ch	103 214	103 114
16 ch	103 216	103 116
18 ch	103 218	103 118

400 cc	Tek Trokar Single Trocar	Çift Trokar Double Trocar
Tip/Type	Ref No	Ref No
8 ch	103 008	103 108
10 ch	103 010	103 110
12 ch	103 012	103 112
14 ch	103 014	103 114
16 ch	103 016	103 116
18 ch	103 018	103 118

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## Drenaj Torbalı Yumuşak Dren

## Soft Drain with Drainage Bag

Ch 10'dan Ch30'a kadar PVC drenli / Ch 22, 24, 27, 30 silikon drenli Ch to 30 ch from - W/ PVC drain / 22, 24, 27, 30 ch - W/ silicone drain

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**PLASTİ-MED Plastik Medikal Ürünler San.Tic.Ltd.fiti**